

### Abutment Screw Torque and Driver

Manufacturer	Torque N-cm							Driver	
	15	20	24	25	30	35	45	Titanium Screw	Gold Screw
Atlantis™ titanium and zirconia abutments utilize the same torque and driver setting as the original implant manufacturer.									
Astra Tech 3.0	X							0.050"- 1.25mm hex	
Astra Tech 3.5/4.0		X						0.050"- 1.25mm hex	
Astra Tech 4.5/5.0				X				0.050"- 1.25mm hex	
BioHorizons® External Hex					X			0.050"- 1.25mm hex	
BioHorizons® Internal					X			0.050"- 1.25mm hex	
BioHorizons® Tapered					X			0.050"- 1.25mm hex	
Biomet 3i™ External Hex						X		0.048" hex	Square
Biomet 3i™ Certain® *		X						0.048" hex	
Biomet 3i™ Low Profile		X						0.048" hex	
Dentsply Friadent			X					0.9mm hex	
Hiossen/Osstem HG mini (3.5)		X						1.2mm hex	
Hiossen/Osstem HG standard (4.0, 4.5, 5.0)					X			1.2mm hex	
Keystone Prime™ (Titanium & Zirconia)					X			Quad	
Keystone Restore®					X			0.048" hex	Square
Neoss Abutment						32		Unigrip	
Neoss Access Prosthetic Screw		X						Unigrip	
NobelBiocare (Active™, Replace™, External Hex)						X		Unigrip	
NobelBiocare Multi-Unit	X							Unigrip	
Straumann® Bone & Tissue Level						X		Star/Torx	
Straumann® Octa Abutment Occlusal Screw	X							Star/Torx	
Straumann® Narrow Neck Occlusal Screw						X		Star/Torx	
Thommen SPI® 3.5	X							Four lobe	
Thommen SPI® 4.0, 4.5, 5.0, 6.0				X				Four lobe	
Thommen SPI® zirconia abutment		X						Four lobe	
Zimmer®					X			0.050"- 1.25mm hex	